

GORE® EXCLUDER® DEVICE FAMILY — SIZING SHEET

Offers a broad range of on-label use for patients needing EVAR

GORE® EXCLUDER® AAA Endoprosthesis featuring C3® Delivery System

Trunk-Ipsilateral Leg Endoprosthesis

Catalogue number	Intended aortic inner diameter (mm)	Aortic Endoprosthesis diameter (mm)	Intended iliac inner diameter (mm)	Iliac Endoprosthesis diameter (mm)	Length to flow divider (cm)	Overall device length (cm)	Recommended sheath size (Fr)
RLT231212	19–21	23	10–11	12	4	12	16
RLT231214	19–21	23	10–11	12	4	14	16
RLT231216	19–21	23	10–11	12	4	16	16
RLT231218	19–21	23	10–11	12	4	18	16
RLT231412	19–21	23	12–13.5	14.5	4	12	16
RLT231414	19–21	23	12–13.5	14.5	4	14	16
RLT231416	19–21	23	12–13.5	14.5	4	16	16
RLT231418	19–21	23	12–13.5	14.5	4	18	16
RLT261212	22–23	26	10–11	12	4	12	16
RLT261214	22–23	26	10–11	12	4	14	16
RLT261216	22–23	26	10–11	12	4	16	16
RLT261218	22–23	26	10–11	12	4	18	16
RLT261412	22–23	26	12–13.5	14.5	4	12	16
RLT261414	22–23	26	12–13.5	14.5	4	14	16
RLT261416	22–23	26	12–13.5	14.5	4	16	16
RLT261418	22–23	26	12–13.5	14.5	4	18	16
RLT281212	24–26	28.5	10–11	12	4	12	18
RLT281214	24–26	28.5	10–11	12	4	14	18
RLT281216	24–26	28.5	10–11	12	4	16	18
RLT281218	24–26	28.5	10–11	12	4	18	18
RLT281412	24–26	28.5	12–13.5	14.5	4	12	18
RLT281414	24–26	28.5	12–13.5	14.5	4	14	18
RLT281416	24–26	28.5	12–13.5	14.5	4	16	18
RLT281418	24–26	28.5	12–13.5	14.5	4	18	18
RLT311413	27–29	31	12–13.5	14.5	5	13	18
RLT311415	27–29	31	12–13.5	14.5	5	15	18
RLT311417	27–29	31	12–13.5	14.5	5	17	18
RLT351414	30–32	35	12–13.5	14.5	6	14	18
RLT351416	30–32	35	12–13.5	14.5	6	16	18
RLT351418	30–32	35	12–13.5	14.5	6	18	18

Contralateral Leg Endoprosthesis

Catalogue number	Intended iliac inner diameter (mm)	Iliac Endoprosthesis diameter (mm)	Contralateral Leg length (cm)	Recommended sheath size (Fr)
PLC121000	10–11	12	10	12
PLC121200	10–11	12	12	12
PLC121400	10–11	12	14	12
PLC141000	12–13.5	14.5	10	12
PLC141200	12–13.5	14.5	12	12
PLC141400	12–13.5	14.5	14	12
PLC161000*	13.5–14.5	16	9.5	12
PLC161200*	13.5–14.5	16	11.5	12
PLC161400*	13.5–14.5	16	13.5	12
PLC181000*	14.5–16.5	18	9.5	12
PLC181200*	14.5–16.5	18	11.5	12
PLC181400*	14.5–16.5	18	13.5	12
PLC201000*	16.5–18.5	20	9.5	12
PLC201200*	16.5–18.5	20	11.5	12
PLC201400*	16.5–18.5	20	13.5	12
PLC231000*	18.5–21.5	23	10	14
PLC231200*	18.5–21.5	23	12	14
PLC231400*	18.5–21.5	23	14	14
PLC271000*	21.5–25	27	10	15
PLC271200*	21.5–25	27	12	15
PLC271400*	21.5–25	27	14	15

Aortic Extender Endoprosthesis

Catalogue number	Intended aortic inner diameter (mm)	Endoprosthesis diameter (mm)	Endoprosthesis length (cm)	Recommended sheath size (Fr)
PLA230300	19–21	23	3.3	16
PLA260300	22–23	26	3.3	16
PLA280300	24–26	28.5	3.3	16
PLA320400	27–29	32	4.5	17
PLA360400	30–32	36	4.5	18

Iliac Extender Endoprosthesis

Catalogue number	Intended iliac inner diameter (mm)	Iliac Endoprosthesis diameter (mm)	Contralateral Leg length (cm)	Recommended sheath size (Fr)
PLL161007	8–9	10	7	12
PLL161207	10–11	12	7	12
PLL161407	12–13.5	14.5	7	12

* PLEASE NOTE:

All 16, 18, 20, 23, 27 mm diameter Contralateral Leg Endoprostheses can be used as Iliac Extenders.

Together, improving life



GORE® EXCLUDER® Conformable AAA Endoprosthesis with ACTIVE CONTROL System

Trunk-Ipsilateral Leg Endoprosthesis

Catalogue number	Intended aortic inner diameter (mm)	Aortic Endoprosthesis diameter (mm)	Intended iliac inner diameter (mm)	Iliac Endoprosthesis diameter (mm)	Length to flow divider (cm)	Overall device length (cm)	Recommended sheath size (Fr)
CXT201212E	16–18	20	10–11	12	5.5	12	15
CXT201412E	16–18	20	12–13.5	14.5	5.5	12	15
CXT231212E	19–21	23	10–11	12	5.5	12	15
CXT231412E	19–21	23	12–13.5	14.5	5.5	12	15
CXT261212E	22–23	26	10–11	12	5.5	12	16
CXT261412E	22–23	26	12–13.5	14.5	5.5	12	16
CXT281212E	24–26	28.5	10–11	12	5.5	12	16
CXT281412E	24–26	28.5	12–13.5	14.5	5.5	12	16
CXT321414E	27–29	32	12–13.5	14.5	6.5	14	18
CXT361414E	30–32	36	12–13.5	14.5	6.5	14	18

Contralateral Leg Endoprosthesis

Catalogue number	Intended iliac inner diameter (mm)	Iliac Endoprosthesis diameter (mm)	Contralateral Leg length (cm)	Recommended sheath size (Fr)
PLC121000	10–11	12	10	12
PLC121200	10–11	12	12	12
PLC121400	10–11	12	14	12
PLC141000	12–13.5	14.5	10	12
PLC141200	12–13.5	14.5	12	12
PLC141400	12–13.5	14.5	14	12
PLC161000*	13.5–14.5	16	9.5	12
PLC161200*	13.5–14.5	16	11.5	12
PLC161400*	13.5–14.5	16	13.5	12
PLC181000*	14.5–16.5	18	9.5	12
PLC181200*	14.5–16.5	18	11.5	12
PLC181400*	14.5–16.5	18	13.5	12
PLC201000*	16.5–18.5	20	9.5	12
PLC201200*	16.5–18.5	20	11.5	12
PLC201400*	16.5–18.5	20	13.5	12
PLC231000*	18.5–21.5	23	10	14
PLC231200*	18.5–21.5	23	12	14
PLC231400*	18.5–21.5	23	14	14
PLC271000*	21.5–25	27	10	15
PLC271200*	21.5–25	27	12	15
PLC271400*	21.5–25	27	14	15

Aortic Extender Endoprosthesis

Catalogue number	Intended aortic inner diameter (mm)	Endoprosthesis diameter (mm)	Endoprosthesis length (cm)	Recommended sheath size (Fr)
CXA200005E	16–18	20	4.5	15
CXA230005E	19–21	23	4.5	15
CXA260005E	22–23	26	4.5	15
CXA280005E	24–26	28.5	4.5	16
CXA320005E	27–29	32	4.5	18
CXA360005E	30–32	36	4.5	18

Iliac Extender Endoprosthesis

Catalogue number	Intended iliac inner diameter (mm)	Iliac Endoprosthesis diameter (mm)	Contralateral Leg length (cm)	Recommended sheath size (Fr)
PLL161007	8–9	10	7	12
PLL161207	10–11	12	7	12
PLL161407	12–13.5	14.5	7	12

* PLEASE NOTE:

All 16, 18, 20, 23, 27 mm diameter Contralateral Leg Endoprostheses can be used as Iliac Extenders.

GORE® EXCLUDER® Iliac Branch Endoprosthesis

Iliac Branch Component

Catalogue number	Intended external iliac vessel diameter (mm)	Diameter iliac end (mm)	Iliac branch component length (cm)	Recommended sheath size (Fr)
CEB231010	6.5–9	10	10	16
CEB231210	10–11	12	10	16
CEB231410	12–13.5	14.5	10	16

Internal Iliac Component

Catalogue number	Intended external iliac vessel diameter (mm)	Diameter iliac end (mm)	Iliac branch component length (cm)	Recommended sheath size (Fr x cm)
HGB161007	6.5–9	10	7	12 x 45
HGB161207	10–11	12	7	12 x 45
HGB161407	12–13.5	14.5	7	12 x 45

 Consult Instructions for Use
eifu.goremedical.com

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. ^{Fr} only

Products listed may not be available in all markets.

GORE, *Together, improving life*, ACTIVE CONTROL, C3, EXCLUDER and designs are trademarks of W. L. Gore & Associates.
© 2021 W. L. Gore & Associates GmbH 2164892-EN1 MARCH 2021

W. L. Gore & Associates, Inc.
goremedical.com

Asia Pacific +65 6733 2882 Australia/New Zealand 1800 680 424 Europe 00800 6334 4673
United States Flagstaff, AZ 86003 800 437 8181 928 779 2771

