


INSTRUCTIONAL LOADING CARD



Cartridge sleeve



Anvil sleeve

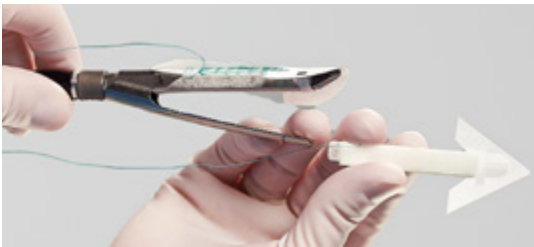

1

Slide the GORE® SEAMGUARD® Reinforcement sleeves onto the appropriate stapler jaws. Position the bioabsorbable material so all of the staple holes and divots are covered.



2

Remove the TYVEK® inserts by separating the distal TYVEK® sections from the underlying tabs, then sliding proximally to release.



3

Ensure that the suture material is not inside the stapler jaw when closing and positioning the stapler. Before firing, pull the suture to release the GORE® SEAMGUARD® Reinforcement devices from the stapler jaws. Fire the stapler.

Designed to work with a variety of staplers

Configured for endoscopic staplers

Stapling device (Gore package label ID)	Cartridge color	Catalogue number
ETHICON ENDOPATH® ETS Flex 45	White, Blue, Green	1BSGETS45E 12BSGETS45E*
ETHICON ENDOPATH® EZ 45	Blue, Green	1BSGEZ45E 12BSGEZ45E*
ETHICON ENDOPATH® ETS Flex 60	White, Blue, Green	1BSGETS60E 12BSGETS60E*
ETHICON ECHELON FLEX 45 ENDOPATH®	White, Blue, Gold, Green	1BSGEC45E 12BSGEC45E*
ETHICON ECHELON FLEX 60 ENDOPATH®	White, Blue, Gold, Green, Black	1BSGEC60AE 12BSGEC60AE*
MEDTRONIC ENDO GIA Universal Roticulator 45	White, Blue	1BSGUSS45BE 12BSGUSS45BE*
MEDTRONIC ENDO GIA Universal Roticulator 45	Green	1BSGUSS45GE 12BSGUSS45GE*
MEDTRONIC ENDO GIA Universal Roticulator 60	White, Blue	1BSGUSS60BE 12BSGUSS60BE*
MEDTRONIC ENDO GIA Universal Roticulator 60	Green	1BSGUSS60GE 12BSGUSS60GE*
Configured specifically for the MEDTRONIC ENDO GIA Reloads with TRI-STAPLE Technology		
MEDTRONIC ENDO GIA Ultra Universal	Purple	1BSGTRI45PE 12BSGTRI45PE*
MEDTRONIC ENDO GIA Ultra Universal	Purple	1BSGTRI60PE 12BSGTRI60PE*
MEDTRONIC ENDO GIA Ultra Universal	Black	1BSGTRI45BE 12BSGTRI45BE*
MEDTRONIC ENDO GIA Ultra Universal	Black	1BSGTRI60BE 12BSGTRI60BE*

* Twelve packages per box.

 Consult Instructions
for Use
eifu.goremedical.com

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. Rx Only

Products listed may not be available in all markets.

ETHICON, ECHELON FLEX and ENDOPATH are trademarks of Ethicon, Inc. a Johnson & Johnson Company.
MEDTRONIC, ENDO GIA and TRI-STAPLE are trademarks of Medtronic, Inc.

GORE, *Together, Improving life*, SEAMGUARD and designs are trademarks of W. L. Gore & Associates.
© 2023 W. L. Gore & Associates GmbH 23858750-EN JANUARY 2023

W. L. Gore & Associates, Inc.
goremedical.com

Asia Pacific +65 67332882 **Europe** 00800 6334 4673
United States Flagstaff, AZ 86003 800 437 8181 928 779 2771

