

SIZING CARD

GORE® VIABAHN® VBX

Balloon Expandable
Endoprosthesis



NOW

6Fr
compatible

GREATER
VERSATILITY

Together, improving life



Device sizing	Introducer sheath (Fr)							
	Stent labeled/ nominal diameter (mm)	Stent length						Maximum post-dilated stent diameter (mm)*
		15 mm	19 mm	29 mm	39 mm	59 mm	79 mm	
5	6	6	6	6	6	6	8	
6	6	6	6	6	6	6	8	
7	6	6	6	6	6	6	11	
8	—	—	7	7	7	7	11	
8L	—	—	7	7	8	8	16 [†]	
9	—	—	7	7	7	7	13	
10	—	—	8	8	8	8	13	
11	—	—	8	8	8	8	16 [†]	

Latest VBX Stent Graft profile (Fr).

All configurations available in .035" over-the-wire 80 cm and 135 cm catheter lengths.

* Secondary balloon required to post-dilate the stent beyond its nominal deployed diameter (Secondary balloon not included).

† Technical limit of the device as determined by in-vitro testing for the indicated use; device expansion beyond 13 mm was not studied as part of the VBX FLEX Clinical Study and is outside of the approved indication — see *Instructions for Use*.



Consult Instructions
for Use

eifu.goremedical.com

INDICATIONS FOR USE IN THE U.S.: The GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis is indicated for the treatment of de novo or restenotic lesions found in iliac arteries with reference vessel diameters ranging from 5 mm–13 mm and lesion lengths up to 110 mm, including lesions at the aortic bifurcation. **CONTRAINDICATIONS:** Do not use the GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis in patients with known hypersensitivity to heparin, including those patients who have had a previous incident of Heparin-Induced Thrombocytopenia (HIT) type II. Refer to *Instructions for Use* at goremedical.com for a complete description of all contraindications, warnings, precautions and adverse events. R_x Only

Products listed may not be available in all markets.

GORE, *Together, improving life*, VBX, VBX FLEX, VIABAHN and designs are trademarks of W. L. Gore & Associates.

© 2020, 2023 W. L. Gore & Associates, Inc. 231096014-EN NOVEMBER 2023

W. L. Gore & Associates, Inc.

goremedical.com

Asia Pacific +65 67332882

Australia/New Zealand 1800 680 424 **Europe** 00800 6334 4673

United States Flagstaff, AZ 86003 800 437 8181 928 779 2771

