


MAKE EARLY CANNULATION COUNT

Three steps for success for the first three sessions within two weeks of graft implantation:

STEPS TO PREP


- Clean gown
- Mask or shield
- Sterile gloves

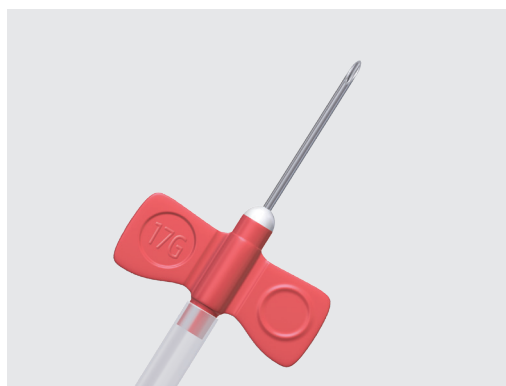

1 | **GO SMALL**
Use a 17 gauge needle to restrict flow



2 | **WATCH FLOW**
Keep the flow rate at 200–250 ml/min to prevent damage to the vein

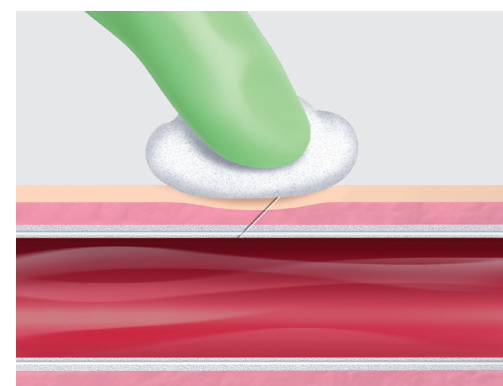


3 | **HOLD PRESSURE**
Hold constant pressure for 10–15 minutes to achieve hemostasis

 Consult Instructions for Use
eifu.goremedical.com

Products listed may not be available in all markets.

GORE, *Together, improving life*, ACUSEAL and designs are trademarks of W. L. Gore & Associates.
© 2021 W. L. Gore & Associates, Inc. 21259592-EN AUGUST 2021

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. 

W. L. Gore & Associates, Inc.
goremedical.com



Asia Pacific +65 67332882 Australia/New Zealand 1800 680 424 Europe 00800 6334 4673
United States Flagstaff, AZ 86003 800 437 8181 928 779 2771

Together, improving life



EARLY CANNULATION AND UNCOMPROMISED HANDLING

Three key steps for implantation:

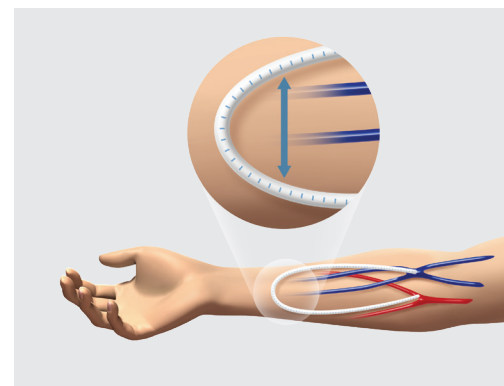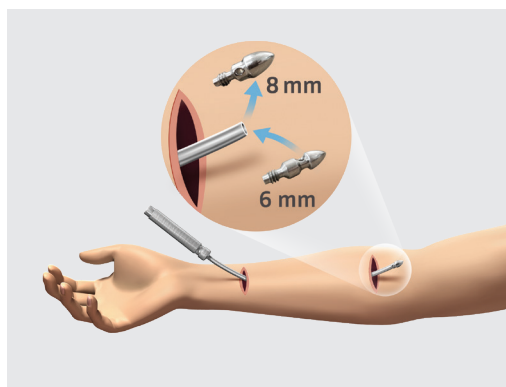

1

CREATE

6 mm graft: tunnel with 8 mm tip, attach graft with 6 mm tip

4–7 mm graft: tunnel with 10 mm tip, attach graft with 7 mm tip

2

CUT

Different tools create different cuts. Scissors are preferred and recommended.

3

SUTURE

Position the graft with a wide curve at the apex

Consult Instructions for Use
eifu.goremedical.com

Products listed may not be available in all markets.

GORE, Together, improving life, ACUSEAL and designs are trademarks of W. L. Gore & Associates.
© 2021 W. L. Gore & Associates, Inc. 21259592-EN AUGUST 2021

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. Only

W. L. Gore & Associates, Inc.
goremedical.com

Asia Pacific +65 67332882 Australia/New Zealand 1800 680 424 Europe 00800 6334 4673
United States Flagstaff, AZ 86003 800 437 8181 928 779 2771

Together, improving life

