



Together, improving life

GORE® Tri-Lobe Balloon Catheter


INFLATION VOLUME GUIDE

BCM1634 – 10 mm Balloon Catheter

| Intended aortic diameter (mm) | Recommended inflation volume (ml) |
|-------------------------------|-----------------------------------|
| 16 | 4 |
| 20 | 6 |
| 24 | 8 |
| 28 | 10 |
| 32 | 13 |

BCL2645 – 16 mm Balloon Catheter

| Intended aortic diameter (mm) | Recommended inflation volume (ml) |
|-------------------------------|-----------------------------------|
| 26 | 12 |
| 30 | 14 |
| 34 | 18 |
| 38 | 21 |
| 42 | 25 |

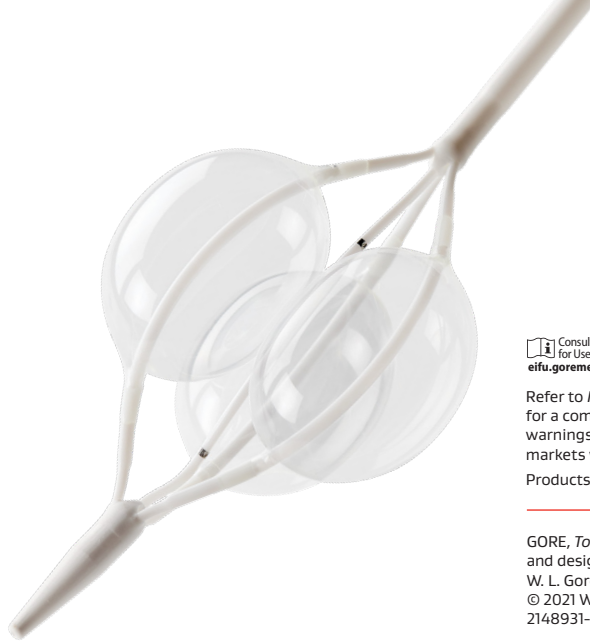

 Consult Instruction
for Use
eifu.goremedical.com

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. Rx Only

Products listed may not be available in all markets.

GORE, *Together, improving life*
and designs are trademarks of
W. L. Gore & Associates.
© 2021 W. L. Gore & Associates, Inc.
2148931-EN JANUARY 2021

