


GORE® EXCLUDER® Thoracoabdominal Branch Endoprosthesis

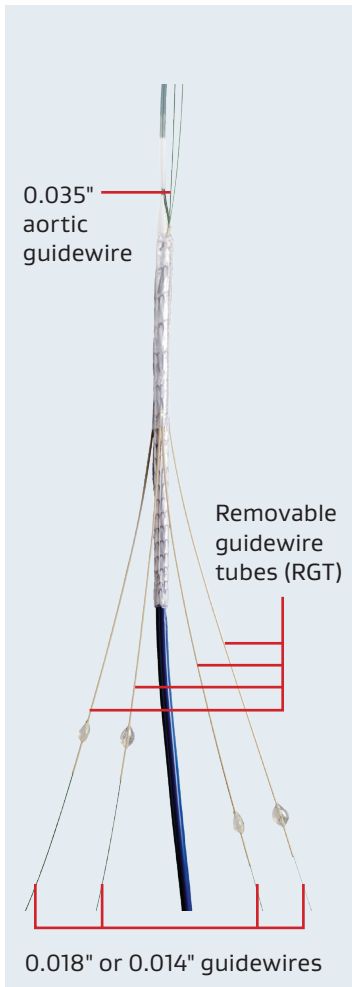


DEPLOYMENT SEQUENCE


1 Establish brachial/axillary (minimum 12 Fr) and femoral (22 Fr) sheaths and 5 through-and-through wires.




2 First stage TAMBE Aortic Component deployment.

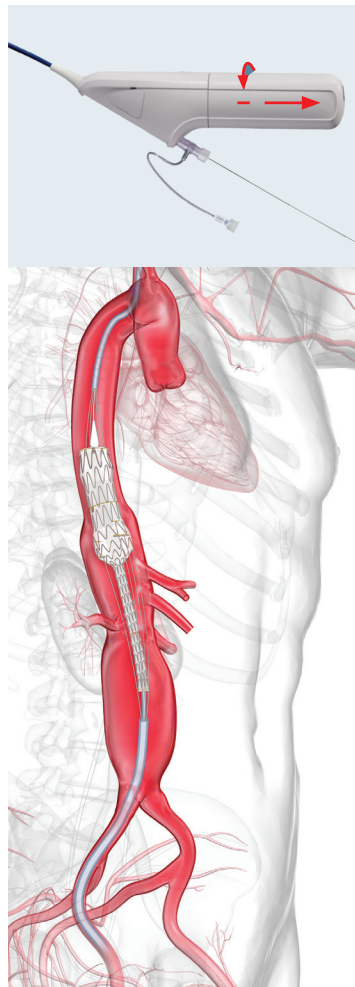


3 Unconstrain proximal anchors and cannulate branch vessels.

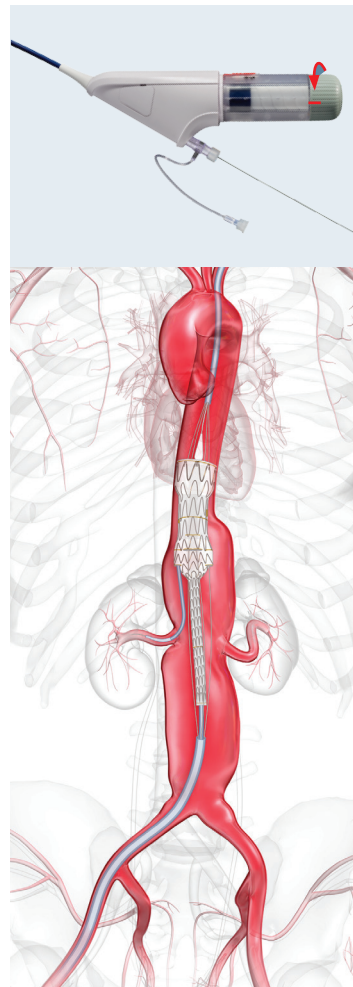


4 Second stage TAMBE Aortic Component deployment.

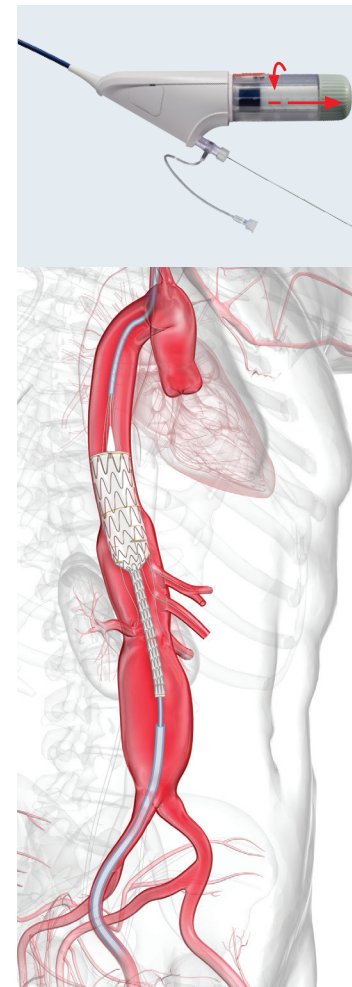


5 Deploy Branch Component in 3 of 4 branches.

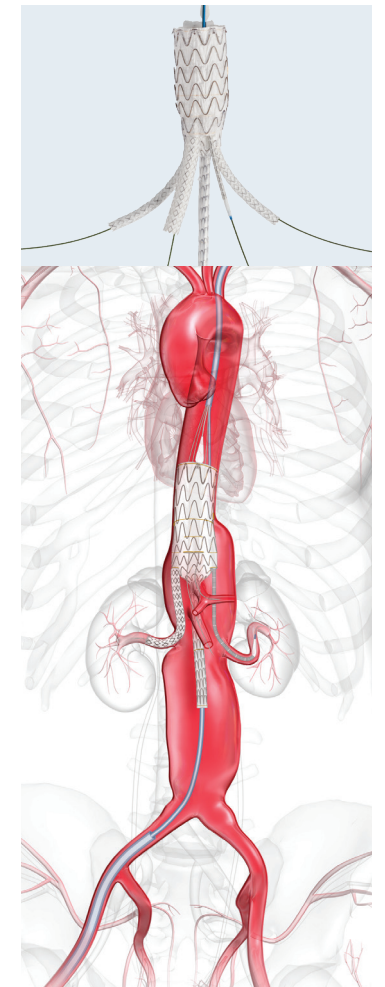


6 Final stage TAMBE Aortic Component deployment.

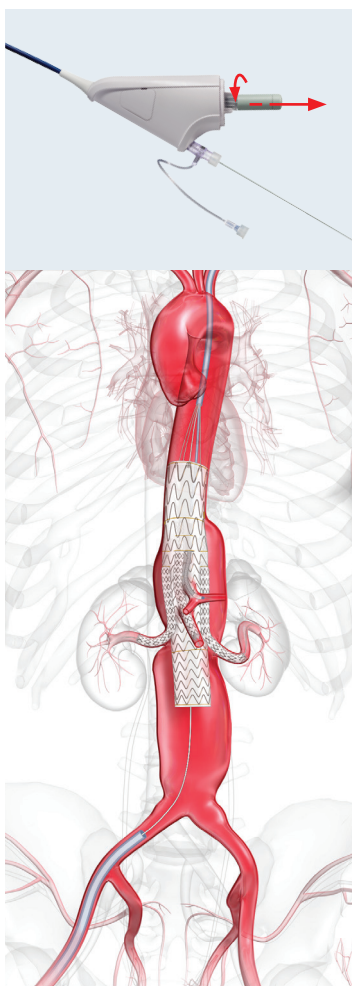


7 Deploy Branch Component in remaining branch.

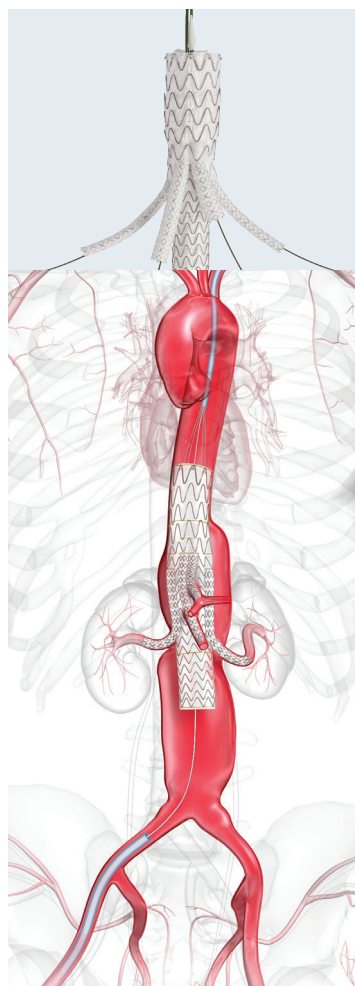


8 First stage Distal Bifurcated Component (DBC) deployment.

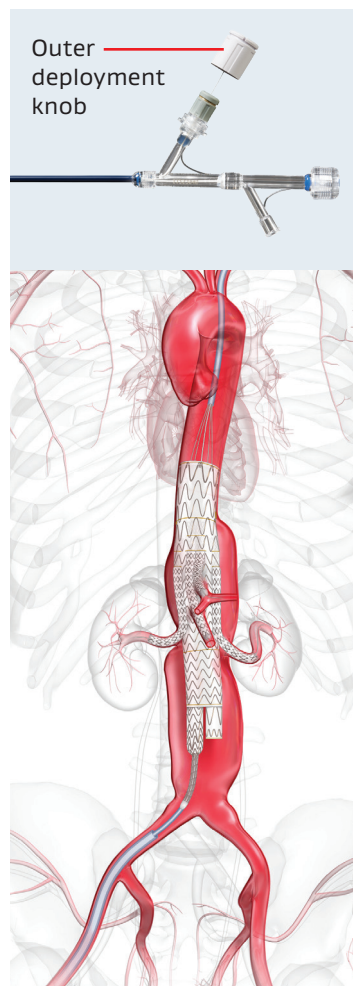

9 Contralateral Leg Component deployment and extension as necessary.

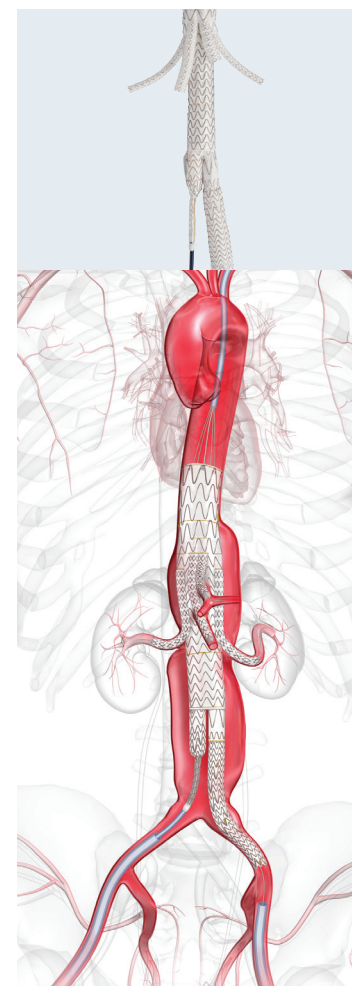

10 Final stage DBC deployment and extension as necessary. Balloon seal zones.

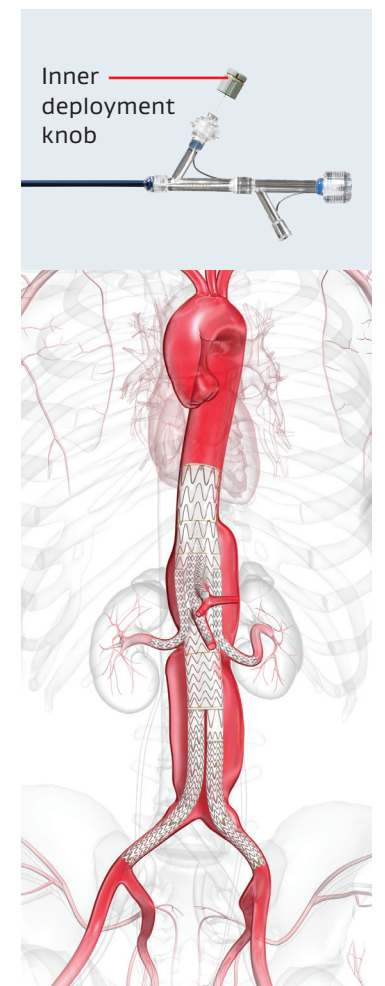

Consult Instructions for Use at eifu.goremedical.com

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. Rx Only

Products listed may not be available in all markets.

GORE, *Together, improving life*, EXCLUDER and designs are trademarks of W. L. Gore & Associates. © 2018, 2024 W. L. Gore & Associates, Inc. 241343653-EN FEBRUARY 2024

W. L. Gore & Associates, Inc.
goremedical.com

Asia Pacific +65 67332882 Australia/New Zealand 1800 680 424 Europe 00800 6334 4673
United States Flagstaff, AZ 86004 800 437 8181 928 779 2771

